

**OLYMPUS**<sup>®</sup>



## Suction Polyp Trap<sup>™</sup>

Simple and easy to use, the Suction Polyp Trap is a reliable solution to capture excised tissue.

# Suction Polyp Trap™



Cat. No. H334 | Suction Polyp Trap | 12 Units/Case

## Features and Benefits

- Clear plastic construction** Allows visual verification of polyp collection.
- Four-chamber indexing system** Permits accurate identification and localization of each polyp.
- Four capture chambers** Offers convenient collection of up to four polyps or tissue specimens, isolating each for identification and pathology study. Reduces contact with patient effluent.
- Easy in-line installation** Installs between the endoscope and the suction receptacle in only a few seconds.
- Individually packaged** Reduces risk of biohazard and sample contamination with single-patient use.
- Lab transport device** Provides easy and safe transport to pathology lab with the sealed unit.

## Installing the Suction Polyp Trap

- 1 Connect tube on center port of trap to suction system.
- 2 Connect outside port to endoscope.
- 3 Rotate jar to align arrow on lid with any open ("0") position for normal suctioning.
- 4 To entrap first polyp, rotate jar and align arrow with number 1.
- 5 After the polyp is trapped, repeat Step 3 for normal suctioning as needed.
- 6 Repeat Step 4 for positions 2, 3, and 4 for each additional polyp (same patient only).
- 7 Follow standard procedures to prepare specimen(s) or simply disconnect tubing from endoscope and suction systems; then remove center tube from Suction Polyp Trap lid.
- 8 Fill Suction Polyp Trap jar with fixative. Replace lid when fixative filling is complete.
- 9 Remove center tube from lid. Bend outer tube to cover both outer and center ports.
- 10 Unit is ready for transport to laboratory.

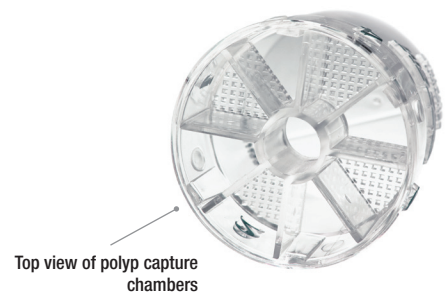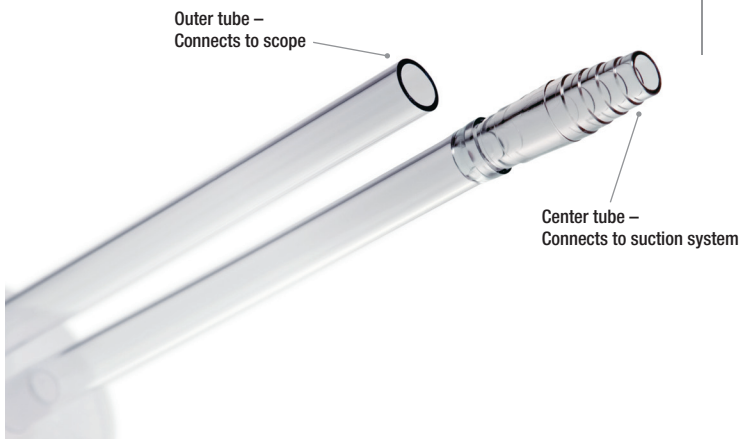

OAIET0110BR06223

To place an order, call **800-848-9024**.  
[www.olympusamerica.com/polyptrap](http://www.olympusamerica.com/polyptrap)

OLYMPUS  
**EndoTherapy**

Manufactured by Bracco Diagnostics Inc.  
 ©2010 Olympus America Inc. All rights reserved.  
 Olympus and Olympus EndoTherapy are registered trademarks of Olympus Corporation, Olympus America Inc., and/or their affiliates. Suction Polyp Trap is a trademark of Bracco Diagnostics Inc.